

PULPDENT

perio
care®

CE
0459

periodontal dressing

Pulpdent Corporation • Watertown, MA • USA

Pulpdent perio care Periodontal dressing

R_x only



- Easy to measure: use equal amounts of paste and gel.
- Easy to mix: paste and gel combine easily and completely.
- Complete: no additional ingredients are necessary.
- Ready to use quickly: 60–90 seconds
- Readily formed into cords of any thickness or length for easy application in the mouth.
- Cohesive: PerioCare melds to itself during working time.

- Adhesive: PerioCare stays in the mouth as long as needed without shrinkage or disintegration.
- Easily removed.
- Naturally colored.
- Neutral aroma and taste.
- Contains no eugenol.
- The package is designed for efficient and convenient office storage. Each tube is clearly marked to avoid confusion between paste and gel.
- Economical for the dentist. Package contains 6 fl. oz. (177.4 ml). A mixing pad is included in the PerioCare kit.

PULPDENT

PerioCare Periodontal Dressing PRODUCT DESCRIPTION

PerioCare is a highly elastic periodontal dressing that sets resiliently hard, does not chip, and provides durable protection of tissue.

INDICATIONS

Post-surgical periodontal dressing for protection of soft tissues

CONTRAINDICATIONS

Not for use on cold sores

DIRECTIONS FOR USE

Helpful Hint: To accelerate setting time, add 1-2 drops of water to PerioCare® paste and gel and incorporate water into the mix.

1. Extrude equal amounts of paste (tube 1) and gel (tube 2) onto mixing pad.
2. Use a spatula to thoroughly mix paste and gel until color becomes uniform throughout.
3. Form PerioCare mixture into log shape on mixing pad.
4. Test for tackiness with spatula or wet finger. PerioCare is ready to use with moistened fingers in about 60-90 seconds.
5. Mold PerioCare into desired shape using fingers. PerioCare will remain workable for approximately 4-5 minutes. If additional working time is required, place PerioCare in hot water to soften.

6. Clean spatula and instruments immediately with Wonder Orange™ solvent or with alcohol.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street

Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.



Leibetseder Dentalwarenhandel GmbH
Hans-Zach-Straße 2
4210 Gallneukirchen
Austria

Bohumila Sieglová
Kolná 303
25203 Řitka
Czech Republic

Hu-Fa Dental a.s.
Moravni 909
765 02 Otrokovice
Czech Republic

Cenger Scandinavia A/S
Urlevvej 6 B
8783 Hornslyd
Denmark

Dansk Nordenta A/S
Nydamsvej 8
8362 Herning
Denmark

Plandent OY
Asentajankatu 6 B
00880 Helsinki
Finland

OPSYSE
18 Allée du Fenouil
ZA Saint Louis
84250 Le Thor
France

KinderDent GmbH
Gutenbergstraße 7
28844 Weyhe
Germany

Ortho Organizers GmbH
Lauenbühlstraße 59
88161 Lindenberg im Allgäu
Germany

Dental Medical Ireland
Kellystown House
Kellystown Lane, Leixlip
Co. Kildare, W23 VF9P
Ireland

Ravelli S.p.A.
Via Darwin 32-34
20019 Settimo Milanese
Italy

Virdental, UAB
Artojų g. 52B, Garliavos Apylinkių
seniūnija
Teleičių kaimas
53273, Kauno Rajonas
Lithuania

Hofmeester Dental B.V.
Andries Copierhof 2
3059 LM Rotterdam
Netherlands

Equadent Sp z o.o.
ul. Pck 12
62-500 Konin
Poland

GAMA Dental AB
Västbergavägen 24
126 30 Hågersten
Sweden

Lifco Dental AB
Verkmästaregatan 1
745 39 Enköping
Sweden



PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.
(617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
www.pulpdent.com • pulpdent@pulpdent.com

XP-PC-IN-07w
REV: 04/2022