PULPDENT®

Sparkle™

Diamond Polishing Paste

PRODUCT DESCRIPTION

Sparkle is specially formulated to produce unsurpassed results on porcelain, gold, metal and composite. It does not splatter, and it washes off easily.

INDICATIONS

For polishing porcelain, gold, metal and composite to a high luster finish

CONTRAINDICATIONS

Not indicated for cleaning enamel

INSTRUCTIONS FOR USE

1. Apply to surface or restoration with brush, rubber cup or felt wheel.
2. Polish to high luster finish.
3. Wash off with water.
4. Keep syringe capped when not in use

CAUTION

Avoid contact with eyes.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, and contamination.

- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
PULPDENT® Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.