
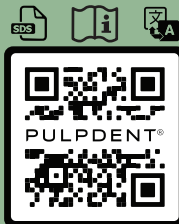
The Original Premixed Calcium Hydroxide Dressing

3mL

Pulpdent® Paste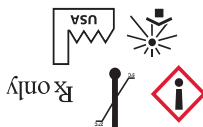

SCAN ME

www.pulpdent.com/info

Pulpdent® Paste

Pulpdent® Paste

For Vital Pulp Therapy and Root Canal Therapy

CE
0459

Pulpdent Paste is the original premixed calcium hydroxide methylcellulose pulpal dressing for vital pulp therapy and root canal therapy. It is radiopaque and requires no mixing.

Pulpdent Paste has a pH >12. It is applied directly on the vital pulp in all pulp capping, pulpotomy and curettage procedures. Pulpdent Paste stimulates the pulp to form a new dentin bridge which is visible clinically after approximately 2–3 months and which re-encloses the pulp in its own natural protective chamber. A hard base must be placed over the dressing prior to the placement of the final restoration.

Pulpdent Paste is a clinically proven dressing for the stimulation of hard tissue formation and root repair in the non-vital root canal. Its uses include temporary canal filling, apexification and other hard tissue formation, and the treatment of traumatic injuries, root resorption, apical cysts and abscesses, periapical lesions, exudation, perforation

and root fractures. Pulpdent Paste discourages traumatic external resorption in the reimplanted avulsed tooth and is ideal as a dressing for weeping canals and after the removal of serous exudate and pus.

INGREDIENTS: Calcium Hydroxide, Aqueous Methylcellulose, Barium Sulfate

PRODUCT DESCRIPTION

Pulpdent Paste is the original premixed calcium hydroxide methylcellulose pulpal dressing for vital pulp therapy and root canal therapy.

INDICATIONS

Direct Pulp Capping, Dentin Bridge Formation, Root Canal Therapy, Apexification

CONTRAINDICATIONS

Not for use as a hard-setting base or hard-setting root canal filling material. For any potential allergies, refer to associated MSDS.

INSTRUCTIONS FOR USE FOR VITAL PULP THERAPY

1. Place an 18 gauge needle securely on the syringe with a twist. The needle can be bent to facilitate access to cavity. Alternatively, dispense a small amount of paste on a pad and place into cavity with an appropriate instrument.
2. For pulpotomy, place 2–3mm of Pulpdent Paste onto pulp. For pulp capping, place 1–2mm of Pulpdent Paste at the exposure site.
3. Dry with stream of air to create a “crust”.
4. Gently tamp Pulpdent Paste with a pledget of cotton to assure contact with the pulp. Again dry with stream of air.
5. Flow hard setting cement over the Pulpdent Paste for compressive strength. Place final restoration.

FOR ROOT CANAL THERAPY

1. Fill root canal with Pulpdent Paste using an 18 gauge Pulpdent Paste needle, the Pulpdent Pressure Syringe, a lentulo spiral or a blunt paper point.
2. Cover with a pledget of cotton and a temporary filling material.
3. Pulpdent Paste can be removed by minimal filing and normal irrigating agents and replaced as necessary until the desired results have been achieved. The root canal can then be obturated by an appropriate method.

NOTE: Use a new applicator tip for each patient. To extend shelf life of Pulpdent Paste, place a drop of water in the cap and recap syringe promptly after each use.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

CAUTION

May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.


REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>
Note: Refer to the Safety Data Sheet for a list of materials present in this product.

CLINICAL SITUATIONS WHERE PULPDENT PASTE CAN BE USED IN ROOT CANAL THERAPY

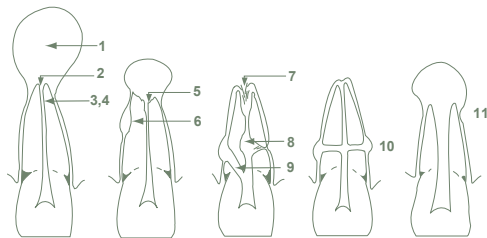1. exudation control
2. large periapical lesions
3. antibacterial dressing
4. temporary root filling
5. apical inflammatory resorption
6. inflammatory resorption following trauma
7. apical internal resorption
8. internal-external root resorption
9. perforations
10. transverse root fractures
11. incompletely developed pulpless teeth (Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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Ravelli S.p.A.
Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB