Embrace™ Esthetic Opaquer

20-Second Light Cure

PRODUCT DESCRIPTION

A thin coat of Embrace Esthetic Opaquer masks metals and discolored tooth surfaces and cures in 20 seconds. Embrace Opaquers behave favorably in the moist oral environment and form chemical bonds to metal, ceramic and tooth surfaces. The material spreads easily with a brush and is available in bleach white, off-white, and pink shades.

INDICATIONS

For masking metal and discolored tooth or restorative surfaces

CONTRAINDICATIONS

Not for direct placement on the exposed pulp

XP-EMOK-IN-06w
REV: 12/2022

INSTRUCTIONS FOR USE

Ceramic and metal surfaces should be dry prior to applying Embrace Opaquer. Leave tooth surfaces slightly moist. Lightly dry and remove surface water with compressed air or a cotton pellet, but do not desiccate the tooth. Tooth surfaces should be shiny or glossy.

- Prepare surfaces as follows:
 - Microabrade **metal and amalgam**.
 - Microabrade **ceramic** or etch with Pulpdent Porcelain Etch Gel for one minute or as indicated by ceramic manufacturer.
 - It is not necessary to etch **dentin or cut enamel**.
- If desired, apply Embrace First-Coat, Pulpdent DenTASTIC UNO or your preferred adhesive to the prepared surface. This step is optional.
- Using a brush or applicator tip, apply a thin coat of Embrace Opaquer into the etched or abraded surface.
- Light cure for 20 seconds.
- Proceed with restoration using your preferred materials.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

CAUTION

Uncured material may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves. Do not exceed manufacturer's recommended curing time for the light you are using.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Ravelli S.p.A.
B.V.**
Lifco Dental AB
UNIDENT AB